



EndoCaddy®



Aztec EndoCaddy® Use & Care Instructions

- EndoCaddy® tubes can be cleaned with mild cleaning detergent and brush, as needed or when solutions are recycled. Rinse tubes thoroughly after cleaning and let dry overnight.
- EndoCaddy® tubes have been used with solutions successfully for years, such as: Ortho-Phthalaldehyde (OPA), Cidex®, Glutaraldehyde, Revital-OX Resert®, Metricide®, Rapicide® and distilled or sterile water (follow the disinfectant manufacturer guidelines for instruction of use).
- **EndoCaddy® tubes are not compatible with Aldahol® or other alcohol containing solutions.**
- Fill with your suggested disinfectant or solution.
- EndoCaddy® tubes are made of Tenite™ Butyrate Clear Plastic (material safety data sheet available upon request)
- Mount EndoCaddy® bracket on wall using 'Toggle Bolts' (2) through 2 small holes in bracket. Place bracket top 36" above floor.
- Aztec recommends end user follow manufacturer cleaning or disinfection guidelines for proper reprocessing of endoscope.

Aztecmed, Inc.

3356 Ironbound Road #303

Williamsburg, VA 23188

800-223-3859 | Aztecmed.com | sales@aztecmed.com